Iso-Form™ Crowns

Indications:

- · Short-term provisional coverage
- Premolars and molars

Tooth Preparation:

Prepare teeth using conventional techniques.

Crown Selection:

· Determine mesio-distal width. Select appropriate crown.



Trial fit for chamfer prep:

• Press gently to stretch crown cervix over finish line of preparation.



Trial fit for shoulder prep:

· Slightly flare the cervix of crown on the stretch block. Do not overexpand.



• Seat expanded crown over prep shoulder.





Iso-Form™ Crowns



Crown Preparation:

 If gingival blanching occurs during trial fit, trim crown using a 3M[™] ESPE[™] Crown Scissors.



• Reset in mouth and have patient establish occlusion.



• Use a sharp explorer to mark the marginal contour. Trim crown back to the line using a 3M[™] ESPE[™] Crown Scissors. Smooth and polish, if desired, using 3M[™] ESPE[™] Sof-Lex[™] Finishing and Polishing Discs and rubber wheel.



• Mix cement and fill crown, assuring voids are not present in cement.

Note: For custom margins and extended durability, line the crown with acrylic or resin material.



Crown Cementation:

 Seat the crown and have the patient bite into occlusion.

Iso-Form™ Crowns

· Burnish the cervical margin to the preparation before the cement sets completely.



· Remove excess cement with scaler or explorer. Floss interproximal areas.

