EXAMIX™ NDS REGULAR TYPE
EXAMIX™ NDS INJECTION TYPE
EXAMIX™ NDS MONOPHASE
EXAMIX™ NDS HEAVY BODY

EXAMIX™ NDS REGULAR TYPE
EXAMIX™ NDS INJECTION TYPE
EXAMIX™ NDS MONOPHASE
EXAMIX™ NDS HEAVY BODY

EXAMIX™ NDS REGULAR TYPE
EXAMIX™ NDS INJECTION TYPE
EXAMIX™ NDS MONOPHASE
EXAMIX™ NDS HEAVY BODY

EXAMIX™ NDS REGULAR TYPE
EXAMIX™ NDS INJECTION TYPE
EXAMIX™ NDS MONOPHASE
EXAMIX™ NDS HEAVY BODY

HYDROPHILIC VINYL POLYSILOXANE IMPRESSION MATERIAL

HYDROPHILES VINYL-POLYSILOXAN ABFORMMATERIAL

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For use only by a dental professional in the recommended indications.

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DESCRIPTION
EXAMIX NDS is a significantly advanced addition reaction (platinum catalyzed) VPS impression material. It produces crisp, extraordinarily smooth and highly precise impressions for models, crowns, bridges and dentures.

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Table with 4 columns: Indication, Impression Technique, Tray Type, Recommended Viscosities. Rows include Single mix single impression, Double mix single impression, Double mix double impression, Double mix single impression, Putty+Wash impression, Impression of partial denture, Impression of full denture.

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CONTRAINDICATIONS
In rare cases, product may cause sensitivity in some people. If any such reactions are experienced, discontinue the use of the product and refer to a physician.

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PROPRIETÀ FISICHE (media)

Table with 5 columns: Test Items, Injection Type 3: Light-Bodied Consistency, Regular Type 2: Medium-Bodied Consistency, Monophase Type 2: Medium-Bodied Consistency, Heavy Type 1: Heavy-Bodied Consistency. Rows include Total Working Time, Minimum Time in Mouth, Recovery From Deformation, Maximum Strain in Compression, Linear Dimensional Change after 24 hrs.

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DIRECTIONS FOR USE

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MODE D'EMPLOI

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Note: The cartridge and mixing tip provided are compatible with GC Cartridge Dispenser 2.

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CARTRIDGE LOADING AND DISPENSING

- 1. Lift the release lever of the CARTRIDGE DISPENSER 2. (referred to as the dispenser hereafter) and pull the piston plunger all the way back into the dispenser. Lift the cartridge holder of the dispenser and load the cartridge, ensuring that the V-shaped notch on the flange of the cartridge is facing down. Push the cartridge holder down to hold the cartridge firmly in place.
2. Lift the release lever and push the piston plunger forward until it engages into the cartridge.
3. Remove the cartridge cap by rotating 1/4 turn anti-clockwise. Tilt the cap downward and peel it away from the cartridge. Gently squeeze the dispenser handle to extrude a small amount of material from the two openings at the end of the cartridge. Make sure that base and catalyst come out evenly.
4. Align the V-shaped notch on the rim of mixing tip to the V-shaped notch between the cartridge barrels. Push firmly to attach the mixing tip. Then rotate the colored collar of the mixing tip 1/4 turn clockwise to the end of the cartridge. The dispenser is now ready for use.
5. Squeeze the handle several times to extrude the material. After use, do not remove the mixing tip as this will become the storage cap until next use. When replacing the mixing tip, rotate the collar on the mixing tip 1/4 turn anti-clockwise to align the V-shaped notch on the cartridge. Tilt the mixing tip downward and peel it away from the cartridge.
6. Remove and replace the old mixing tip immediately prior to next use. Before attaching a new tip, gently extrude a small amount of material to ensure that base and catalyst are flowing evenly from both openings. If the materials should fail to extrude, remove any hardened materials from the end of the cartridge.
7. To replace the cartridge, lift the release lever and retract the piston plunger fully. Remove the empty cartridge by lifting the cartridge holder and load a new one into the dispenser.

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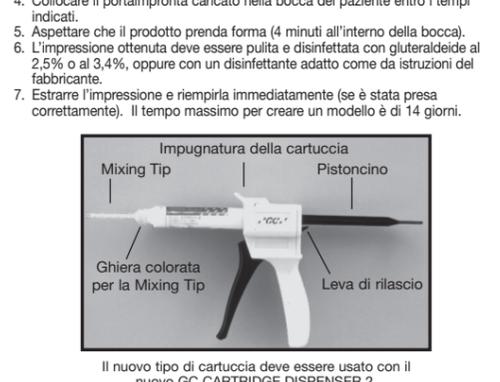
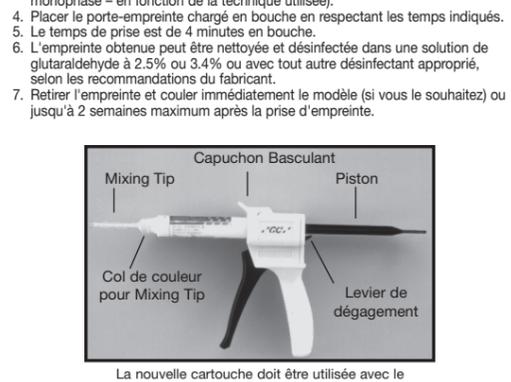
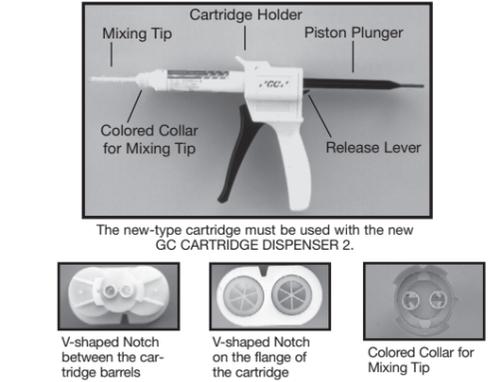
USO DEL MATERIALE DA IMPRONTA

Note: Prior to use, leave the material to stand at room temperature. Cooled or refrigerated material may delay setting.
1. Attach an intraoral tip to the mixing tip or load material into a syringe.
2. Inject the mixture onto the prepared teeth.
3. Load the tray with the appropriate material (putty, monophasic, heavy body or regular - depending on technique being used).
4. Seat the loaded tray in the mouth within specified working times.
5. Wait for set (4 minutes in the mouth).
6. The obtained impression should be cleaned, then disinfected, utilizing a 2.5% or 3.4% glutaraldehyde, or other appropriate disinfectant, according to the manufacturers label recommendations.
7. Remove the impression and pour a model immediately (if desired). Maximum time for pouring the model is 14 days.

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- NOTES
1. When extruding EXAMIX NDS, care should be taken to avoid mixing or contact with the following materials. They may delay or prevent setting:
• Catalyst for condensation type silicone impression materials
• Polysulfide impression materials
• Eugenol materials
• Sulfur
• Latex
• Oil
• Acrylates
Also avoid moisture and glycerol when extruding.
2. Avoid use of EXAMIX NDS with patients who have a history of hypersensitivity to silicone impression materials.
3. EXAMIX NDS can be silver- or copper-plated.
4. Care should be taken to avoid getting material on clothing. It is hard to remove.

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LAGERUNG

STOCKAGE

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Store in a dry location at normal room temperature. (74°F/23°C) (Relative humidity: 50%). Avoid direct sunlight. Shelf life: 2 years from date of manufacture.

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PRESENTATION

CONFEZIONI

- Note: All packaging types may not be available in all countries.
EXAMIX NDS 2 Cartridge Package with Tips
-Regular -Monophase
-Injection -Heavy
EXAMIX NDS 32 Cartridge Package Refill without Tips
-Regular -Monophase
-Injection -Heavy
EXAMIX NDS 80 Cartridge Package Refill without Tips
-Regular -Monophase
-Injection -Heavy

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ATTENTION

ATTENZIONE

- 1. Avoid contact of base, catalyst, mixed material or adhesive with eyes. In case of contact, immediately flush with generous amount of water and seek medical attention.
2. Universal VPS Adhesive (optional) is flammable. Do not use or store near heat or open flame. Use in a well ventilated area.
3. Avoid getting material on clothing as it is hard to remove.

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Last revised: 10/2010

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Ultima revision: 10/2010

DISTRIBUTED by GC CORPORATION
76-1 Hasunuma-cho, Itabashi-ku, Tokyo 174-8585, Japan
Tel: +81-3-3965-1221

EU: GC EUROPE N.V.
Interleuvenlaan 33, B-3001 Leuven, Belgium
Tel: +32-16-74.10.00

GC AMERICA INC.
3737 West 127th Street, Alsip IL 60803 U.S.A.
Tel: +1-708-597-0900

GC ASIA DENTAL PTE. LTD.
19 Loyang Way, #06-27 Singapore 508724
Tel: +65-6546-7588 3021010

EXAMIX™ NDS REGULAR TYPE
EXAMIX™ NDS INJECTION TYPE
EXAMIX™ NDS MONOPHASE
EXAMIX™ NDS HEAVY BODY

HYDROPHILIC VINYL POLYSILOXANE IMPRESSION MATERIAL

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