


Sof-Lex™

Finishing and Polishing System Procedure Guide

Sof-Lex System Procedure

The Sof-Lex System is color-coded from dark (coarse) to light (superfine) for an easily followed step-by-step process. Easy, pop-on mounting lets you quickly change discs or spiral wheels—so you can move efficiently through your finishing and polishing procedure.

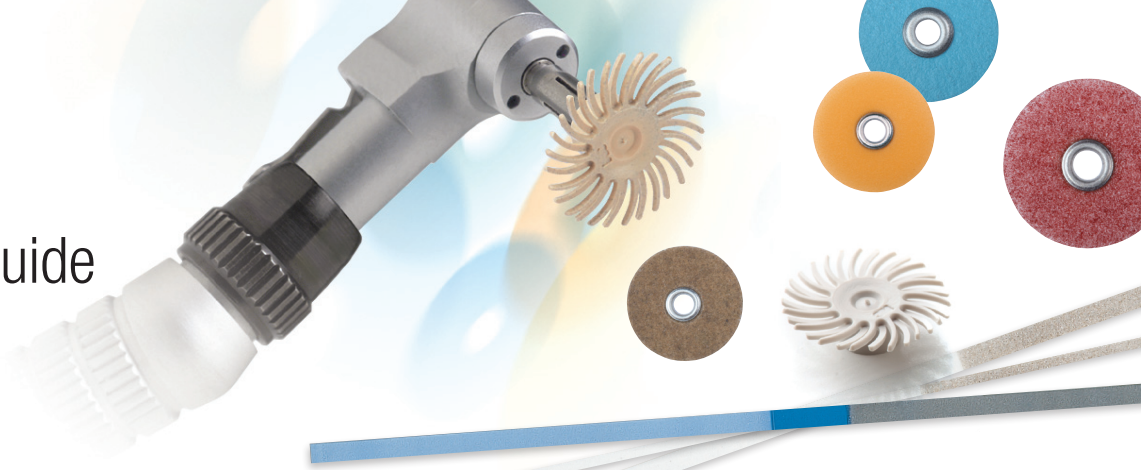

	Sof-Lex™ Discs	Sof-Lex™ XT Discs	Sof-Lex™ Strips	Sof-Lex™ Spiral Wheels	Anterior	Posterior	Interproximal
Gross Reduction	<ul style="list-style-type: none"> Excess removal Shape anatomy Remove air-inhibited layer   	<ul style="list-style-type: none"> Excess removal Shape anatomy Remove air-inhibited layer   	<ul style="list-style-type: none"> Excess removal Shape anatomy Remove air-inhibited layer 	<ul style="list-style-type: none"> Excess removal Shape anatomy Remove air-inhibited layer 	<ul style="list-style-type: none"> Excess removal Shape anatomy Remove air-inhibited layer Sof-Lex™ Discs, Coarse Rotating coarse or medium contouring instruments (e.g., diamonds, carbide burs, stones)	<ul style="list-style-type: none"> Excess removal Shape anatomy Remove air-inhibited layer Rotating coarse or medium contouring instruments (e.g., diamonds, carbide burs, stones)	<ul style="list-style-type: none"> Excess removal Shape anatomy Remove air-inhibited layer Sof-Lex™ XT Discs, Coarse Sof-Lex™ Strips • Coarse/Medium • Coarse/Medium, Narrow
Final Contour	<ul style="list-style-type: none"> Reduce surface roughness Complete and refine contours and margins   	<ul style="list-style-type: none"> Reduce surface roughness Complete and refine contours and margins   	<ul style="list-style-type: none"> Reduce surface roughness Complete and refine contours and margins 	<ul style="list-style-type: none"> Reduce surface roughness Complete and refine contours and margins 	<ul style="list-style-type: none"> Reduce surface roughness Complete and refine contours and margins Sof-Lex™ Discs, Medium Rotating medium or fine contouring instruments (e.g., yellow or red ring diamonds)	<ul style="list-style-type: none"> Reduce surface roughness Complete and refine contours and margins Rotating medium or fine contouring instruments (e.g., yellow or red ring diamonds)	<ul style="list-style-type: none"> Reduce surface roughness Complete and refine contours and margins Sof-Lex™ XT Discs, Medium Sof-Lex™ Strips • Coarse/Medium • Coarse/Medium, Narrow
Finish	<ul style="list-style-type: none"> Remove scratches Smooth surface Pre-polish   	<ul style="list-style-type: none"> Remove scratches Smooth surface Pre-polish   	<ul style="list-style-type: none"> Remove scratches Smooth surface Pre-polish 	<ul style="list-style-type: none"> Remove scratches Smooth surface Pre-polish 	<ul style="list-style-type: none"> Remove scratches Smooth surface Pre-polish Sof-Lex™ Spiral Finishing Wheel Sof-Lex™ Discs, Fine	<ul style="list-style-type: none"> Remove scratches Smooth surface Pre-polish Sof-Lex™ Spiral Finishing Wheel	<ul style="list-style-type: none"> Remove scratches Smooth surface Pre-polish Sof-Lex™ Spiral Finishing Wheel Sof-Lex™ XT Discs, Fine Sof-Lex™ Strips, Fine/Superfine
Polish	<ul style="list-style-type: none"> Final polish to high gloss   	<ul style="list-style-type: none"> Final polish to high gloss   	<ul style="list-style-type: none"> Final polish to high gloss 	<ul style="list-style-type: none"> Final polish to high gloss 	<ul style="list-style-type: none"> Final polish to high gloss Sof-Lex™ Spiral Polishing Wheel Sof-Lex™ Discs, Superfine	<ul style="list-style-type: none"> Final polish to high gloss Sof-Lex™ Spiral Polishing Wheel	<ul style="list-style-type: none"> Final polish to high gloss Sof-Lex™ Spiral Polishing Wheel Sof-Lex™ XT Discs, Superfine Sof-Lex™ Strips, Fine/Superfine

3M ESPE

Dental Products

2510 Conway Avenue
 St. Paul, MN 55144-1000 USA
 1-800-634-2249

★★★★★ Review these products at www.3MESPE.com/review

Learn more at www.3MESPE.com/SofLex

Please recycle. Printed in U.S.A.
 70-2013-0564-9

3M, ESPE and Sof-Lex are trademarks of 3M or 3M Deutschland GmbH. Used under license in Canada. © 3M 2013. All rights reserved.