Permadyne™

en Polyether Impression Material

de Polyether Abformmasse

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Dental Products

St. Paul, MN 55144-1000 U.S.A.

3M ESPE Customer Care/MSDS Information:

U.S.A.: 1-800-634-2249 and Canada 1-888-363-3685.

3M, ESPE and Permadyne are trademarks of 3M Company or

3M Deutschland GmbH. Used under license in Canada.

© 2012, 3M. All rights reserved.

Information valid as of December 2011

4400818142701

CE

ENGLISH

Product Description

Light-bodied and heavy-bodied consistency Permadyne are polyether impression materials for manual mixing. The mixing ratio is 7 volumes base paste : 1 volume catalyst.

For details on all products mentioned below please refer to the corresponding Instructions for Use. Instructions for Use should not be discarded for the duration of product use.

Indications

- Impression of inlay, onlay, crown, bridge, and veneer preparations
- Functional impressions

Precautionary Measures

If an adverse reaction is observed in a patient, discontinue use of the product in this patient.

3M MSDS can be obtained from www.mmm.com or contact your local subsidiary.

Preparation

Block-out undercuts or areas where gingival recession exists to prevent the impression material from "locking" onto the tooth structure. Failure to block-out may make tray removal difficult, or cause extraction of natural teeth or prosthesis. The block-out is especially important when using impression materials with high Shore hardness.

Impression tray:

Any impression trays generally used for precision impressions are suitable. For sufficient adhesion, apply a thin layer of Polyether Adhesive, to the tray and allow to dry completely (at least 30–60 sec, 15 min drying time is ideal).

Retraction

Suitable retraction agents are e.g., aluminum hydrochloride- and aluminum sulfate-based solutions. Retraction with threads or rings containing epinephrine (adrenaline), 8-hydroxyquinoline sulfate or iron (III) sulfate may impair the setting of polyether impression materials.

Keep the area dry from which an impression is to be taken.

Hemostatic threads or rings may be used for subgingival preparations.

Thoroughly remove the residue of the retraction agent by rinsing and drying.

Dosing and Mixing

Dispense equal lengths of light-bodied or heavy-bodied consistency base paste and catalyst onto two separate mixing blocks. Over- and under-dosing of the catalyst does not affect processing time, but impairs the quality of the impression.

In mixing the pastes, touch off at the edge of the block unmixed quantities of paste adhering to the spatula, and repeatedly take up and streak out until the colour of the mixture is uniform. Do not mix by stirring in order to avoid the formation of bubbles!

Times

Mixing time from start of mixing min:sec

Processing time from start of mixing min:sec

Setting time in the mouth min:sec

Residence time in the mouth min:sec

These are the processing times at 23°C/74°F. Higher temperatures shorten, and lower temperatures prolong, total processing times.

Impression Taking

Double-mixing technique

Initially mix the light-bodied consistency paste, and use the filling device to fill the mixture into the Elastomer syringe.

Apply the mixture into the sulcus or cavity from bottom up, always keeping the tip of the application nozzle immersed in the material and in contact with the surface of the tooth.

During application around the preparation, mix the heavy-bodied consistency as described above, and load on the tray previously prepared with adhesive.

Immediately after application around the preparation, position the loaded tray in the mouth, otherwise the syringe material will set faster than the tray material resulting in distortion of the impression. Keep the tray in the mouth steady without applying pressure until setting is complete.

To remedy initial adhesion ("setting the valve"), especially with upper jaw impressions, remove the tray from the gingiva on one side in a posterior position. If this proves difficult it may be necessary to carefully blow some air or water between the impression and the gingiva.

Functional Impression

After preparing the custom tray with adhesive, load the heavy-bodied consistency onto the tray, place the tray in the desired location in the mouth, and have the patient perform functional excursions.

If required, use the light-bodied consistency for precise impression taking (correction).

After Impression Taking

Thoroughly examine and explore the sulcus of the prepared teeth and surrounding dentition. Remove any residual cured impression material from the mouth.

Hygiene

Place the impression in a standard disinfectant solution commonly used for impressions, for the period of time recommended by the manufacturer. Excessive disinfection may damage the impression.

After disinfection, rinse the impression under running water for approx. 15 sec.

Model Preparation

Prepare a cast from the impression with a commercial specialised stone plaster no earlier than 30 min and no later than 14 days after taking the impression.

To avoid introducing bubbles in the model, briefly pre-rinse with water and dry with air before pouring. Do not use surfactants as these impair the quality of polyether impressions, and are not required.

Polyether impressions can be silver-coated, whereas copper-coating is not feasible.

Cleaning

- Paste that has not set may be removed with ethanol or by rinsing with water and soap.
- The adhesive can be removed from metal trays with acetone.

Notes

- At temperatures below 18°C/64°F, the pastes cannot be extruded from the tube due to their increased viscosity. However, at room temperature processability is restored without compromising quality.
- Direct exposure to sunlight and damp storage conditions may damage the impression.
- Polyether impressions should not be exposed to solvent-containing liquids, as this may result in swelling and imprecise modelling.
- 3M ESPE polyether materials may only be combined with 3M ESPE polyethers. The working and setting times of the individual products must be strictly observed.

Storage and Stability

Store the product at 18–25°C/64–77°F. Do not refrigerate!

Do not use after the expiration date.

Store impressions dry and below 30°C/86°F in the dark.

Customer Information

No person is authorized to provide any information which deviates from the information provided in this instruction sheet.

Warranty

3M Deutschland GmbH warrants this product will be free from defects in material and manufacture, 3M Deutschland GmbH MAKES NO OTHER WARRANTIES INCLUDING ANY IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. User is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusive remedy and 3M Deutschland GmbH's sole obligation shall be repair or replacement of the 3M Deutschland GmbH product.

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