

- Remove any remaining uneven areas on the edges of restorations and polish the marginal area.
- Check occlusion.

#### **After Shaping**

- Carefully check the sulcus of the treated teeth and the surrounding areas, remove any cement residues still remaining.

**SHELF LIFE:** The lot number & expiry date are indicated on the product. Do not use after expiry date.

**PRECAUTIONARY INFORMATION FOR PATIENTS:** Avoid use of this product in patients with known methacrylate allergies. If prolonged contact with oral soft tissues occurs, flush with large amounts of water.

#### **PRECAUTIONARY INFORMATION FOR DENTAL PERSONNEL:**

ProLink Cem contains methacrylate monomers that may cause an allergic reaction by skin contact in certain individuals. To reduce the risk of allergic response, minimize exposure to these materials. In particular, avoid exposure to uncured paste. If skin contact occurs, wash immediately with soap and water.

The use of protective gloves and a no-touch technique is recommended. Acrylates may penetrate commonly used gloves. If product contacts the glove, remove and discard the gloves. Wash your hands immediately with soap and water and re-glove. If allergic reaction occurs, seek medical attention as needed.

**CAUTION:** US Federal Law restricts this device to sale or use on the order of a dental professional.

**EU Notice:** Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

**STORAGE & DISPOSAL:** Store in a cool and dark place / room temperature (5-30°C / 41-86°F) away from direct sunlight.

Dispose of empty syringes in accordance to local regulations.

See the Safety Data Sheet (available at [www.silmetdental.com](http://www.silmetdental.com) or through your local subsidiary) for disposal & safety information.

**WARRANTY:** Silmet Ltd. will replace product that is proven to be defective. Silmet Ltd. does not accept liability for any damage or loss, direct or consequential, arising from the use of or inability to use the product described. It is the responsibility of the dentist to determine before use, the suitability of the product for its intended use. The dentist assumes all risk and liability in connection therewith.

Pour des instructions dans d'autres langues

Die Anleitung in weiteren Sprachen:

Para obtener las instrucciones en otros idiomas:

Per istruzioni in altre lingue

FR

DE

ES

IT

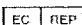
[www.silmetdental.com/downloads/downloads\\_instruction](http://www.silmetdental.com/downloads/downloads_instruction)

 Silmet Ltd.

12 Hassadna st., Or-Yehuda, 6022011, Israel

Tel: 972-3-7353000, Fax: 972-3-5331581

info@silmet.co.il, [www.silmetdental.com](http://www.silmetdental.com)

 Obelis s.a

Bd. General Wahis 53. 1030 Brussels, Belgium

Tel: + (32) 2.732.59.54 Fax: + (32) 2.732.60.03,

E-Mail: mail@obelis.net

 **SILMET**