



## Advanced Fluoride Varnish

Provides increased fluoride uptake!<sup>1</sup>



# Enamel Pro® Varnish

The first advanced fluoride varnish optimized to provide valuable tooth building ions of calcium and phosphate



### Indications For Use



**Enamel Pro Varnish** is a fluoride-containing preparation for the treatment of dentinal hypersensitivity and for the reduction of post-operative sensitivity.

Enamel Pro Varnish offers a nice esthetic appearance. It is patient pleasing and has studies supporting fluoride uptake, fluoride release and diminished hydraulic conductance.<sup>1,2,3</sup>



### Easy Application

- Peel off foil top, stir varnish with applicator brush, mix thoroughly
- Apply thin coat of varnish across tooth surfaces
- Sets on contact with saliva
- Provides increased fluoride uptake into sound enamel and surrounding lesioned enamel (See fluoride uptake chart on back.)<sup>1</sup>



# Enamel Pro® Varnish

Protect your patients' teeth!




**Enamel Pro Varnish** with ACP formula delivers more enamel fluoride uptake and greater fluoride release than 3M Vanish with TCP!<sup>1</sup>

Enamel Pro Varnish provides 4x more fluoride uptake into sound enamel and significantly more fluoride uptake in surrounding lesioned enamel than 3M Vanish.<sup>1</sup> In an independent study evaluating incubation conditions on fluoride release in vitro, Enamel Pro Varnish displayed greater fluoride release than both 3M Vanish and Colgate Duraphat.<sup>1,2</sup>


## Enamel Fluoride Uptake<sup>1</sup>

Post-treatment Fluoride (ppm)

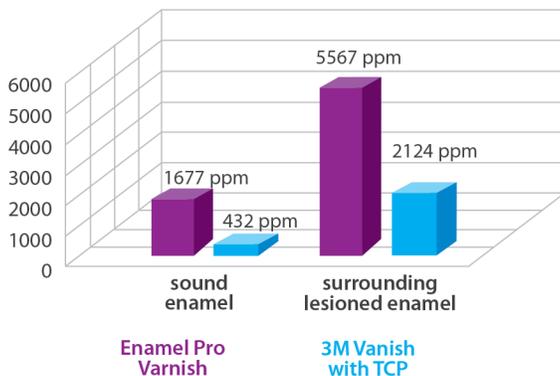

## Fluoride Released (Accumulated Percentage)<sup>2</sup>

Artificial Saliva/Body Temperature 36°C/96.8°F

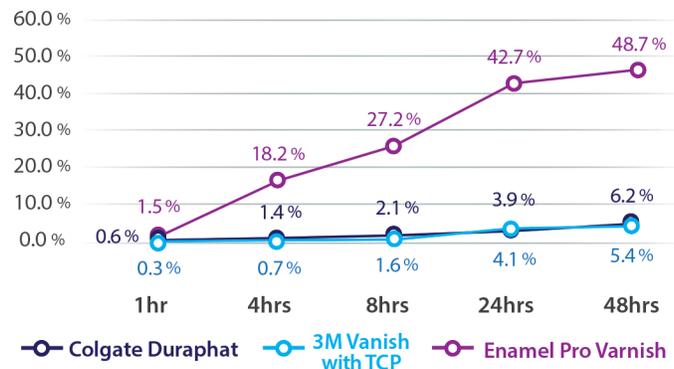

## Ordering Information

### 5% NaF Varnish - Gluten FREE!

#### Box of 35

- 9007540 Strawberries 'n Cream 0.40ml
- 9007541 Bubblegum 0.40ml
- 9007547 VanillaMint 0.40ml
- 9007542 Bubblegum 0.25ml

#### Box of 200

- 9007543 Strawberries 'n Cream 0.40ml
- 9007544 Bubblegum 0.40ml
- 9007545 VanillaMint 0.40ml



**View Easy Application Video:**  
Scan the QR code provided or visit  
[www.youtube.com/PremierDentalProduct](http://www.youtube.com/PremierDentalProduct)

## Practice Building

**Enamel Pro Varnish** provides rapid relief to sensitivity with an in-office application. To maximize patient benefit, dispense **Enamelon® Preventive Treatment Gel** for at-home use and ongoing sensitivity relief. This prevention protocol can help treat sensitivity, prevent caries and gingivitis, while strengthening enamel, and also moisturizes and lubricates dry mouth tissues.<sup>4</sup>

9007285 **Enamelon® Preventive Treatment Gel**  
4.0 oz (113 g), Clean Mint, 12/Case

**Learn More About Enamelon:**  
Contact your Premier sales consultant



**Premier® Dental Products Company** • 1710 Romano Drive, Plymouth Meeting, PA 19462, USA  
Phone 888-670-6100 • 610-239-6000 • Fax 610-239-6171 • [www.premusa.com](http://www.premusa.com) • E-mail: [dentalinfo@premusa.com](mailto:dentalinfo@premusa.com)

1. Schemehorn, B.R., Wood, G.D., McHale, W., Winston, A.E., "Comparison of Fluoride Uptake into Tooth Enamel from Two Fluoride Varnishes Containing Different Calcium Phosphate Sources", J Clin Dent, 2011;22:51-54 / 2. Fluoride Release from Fluoride Varnishes is Affected by Incubation Conditions C. Gonzalez-Cabezas, S.E.Flannagan, R. Krell, M. Uekihara, and C.C. Niquette, Dent Res 91(Spec Iss A):1094, 2012 ([www.dentalresearch.org](http://www.dentalresearch.org)). / 3. "Effect of Calcium Phosphate Containing Fluoride Varnish of Dentin Permeability", Tung, M.S., Torres, J., (ADAHF Paffenbarger Research Center, NIST, Gaithersburg, MD and U.S. Navy Dental Corps, Bethesda MD), J Dent Res (Spec Iss A): 0985, 2008 ([www.dentalresearch.org](http://www.dentalresearch.org)). / 4. Data on file. / † 3M Vanish and Colgate Duraphat are not trademarks of Premier Dental. / 8698761 081678 Rev12 NKP10M