

Film Processor Maintenance

You depend on radiographic films for diagnosis and often employ them in the presentation of treatment plans or as a patient education aid. You realize the importance of consistent, artifact free film processing.



Regular Maintenance of your processor with the right maintenance products ensures best quality radiographs and longest service life.

Air Techniques' maintenance products are specially formulated for maximum cleaning efficiency and ease-of-use.

Build-up residues are easily removed and staining is reduced; protecting films from roller deposits.

All Air Techniques' maintenance products are environmentally safe, they do not contain any heavy metals.

NEW Formula 2000PLUS

The non-toxic and non-hazardous cleaning concentrate effectively removes deposits and stains from roller transports, processors and dip tanks.



**AIR
TECHNIQUES**
INC.

Maintenance for Roller Transport Processors with New & Improved Formula 2000PLUS and Spray 2000PLUS



Quarterly Cleaning with Formula 2000PLUS

Use Formula 2000PLUS for quarterly cleaning to reduce staining and prolong the life of your processor. Formula 2000PLUS is a remarkable two-component formulation. The non-toxic and non-hazardous Formula 2000PLUS effectively dissolves deposits, dirt and algae; all residues of film processing. The deep cleaning of your Roller Transport Processors takes only 15 minutes.

Weekly Cleaning with Spray 2000PLUS

Use SPRAY 2000PLUS for the weekly cleaning of roller processors and dip tanks to remove residue and protect films from roller deposits. Spray 2000PLUS is available in a convenient, ready to use spray bottle. No mixing is necessary, just spray, sponge and rinse under running water. Use Spray 2000 for your A/T 2000, A/T 2000Plus and A/T 2000XR.

Maintenance for Peri-Pro Transports with Peri-Pro Transport Cleaning Kit



Peri-Pro Transport Cleaning Kit with Formula 2000PLUS

Use the Peri-Pro Transport Cleaning Kit for the quarterly cleaning of Peri-Pro Transports to reduce staining, decrease normal maintenance time and prolong the life of your Peri-Pro. Peri-Pro Transport Cleaning Kit is an easy and convenient way to care for your Peri-Pro. It is specially designed to hold the transport with just the right amount of Formula 2000PLUS and water. The tub is completely self-contained, portable and requires little counter space. The deep cleaning with Peri-Pro Transport Cleaning Kit takes only 30 minutes. The Peri-Pro Transport Cleaning Kit can be used for all models of Peri-Pro.