A background pattern of light gray hexagons of varying shades, arranged in a honeycomb-like structure.

Herculite[®] *Ultra*

Nanohybrid Composite Restorative

Instruction For Use

Herculite Ultra

Nanohybrid Composite Restorative

Nanohybrid composite Herculite Ultra is a light-cured, resin-based, dental restorative designed for direct placement. It is indicated for all cavity classes in anterior and posterior teeth.

Herculite Ultra offers superior aesthetics, handling and mechanical properties for simplified restoration procedure without sacrificing performance.

Herculite Ultra is easy to polish, retains clinical luster, and has a lifelike appearance replicating fluorescence and opalescence of the natural tooth. We have taken extraordinary steps to maintain the mechanical properties, fracture toughness and wear resistance, giving you the durability and reliability you have come to expect from the Herculite brand.

Storage Conditions and Shelf Life:

Herculite Ultra should be stored at ambient temperature.

Any packaging containing OptiBond All-In-One should be refrigerated 36-46°F (2-8°C).

The expiration date is marked on packaging. Do not use after expiration date.

Every effort should be made to protect the composite from visible light prior to use.

PLEASE NOTE THE FOLLOWING:

You have purchased this composite in either a bulk syringe or Unidose® delivery. Either delivery has certain handling and storage requirements that if followed will provide excellent performance for the user over the life of the material. Please follow the appropriate recommendations indicated below for the delivery choice that you have made.

Syringe Delivery: To ensure optimal performance of syringes during use: After dispensing the composite, relieve pressure by turning the syringe handle back two turns. This will prevent material build-up in the cap and provide reliable composite use after storage.

Unidose® Delivery: Unidose® tips are designed for SINGLE PATIENT USE ONLY. Do not re-cap and/or re-use the Unidose® tip once material has been dispensed for that patient.

PRIOR TO PLACEMENT -- RECOMMENDATIONS ON PROPER BONDING

- Isolate tooth from adjacent teeth. Use clear, thin strips. Rubber dam is highly recommended.

- Moist dentin surfaces (rather than dry, desiccated dentin) have been shown to enhance the bond strengths of today's modern bonding agents (such as OptiBond All-In-One, OptiBond Solo Plus, or OptiBond FL). NOTE: "Moist" does not infer blood or saliva contamination.
- Placement sequences such as those outlined in this instruction booklet should be followed closely.
- Take a moment to identify the surface to which you are bonding.
- Please take care to ensure that your air line is free of oil and other contaminants.

OPTIBOND ALL-IN-ONE

If you have purchased Herculite Ultra in kit form, you will notice that OptiBond All-In-One is included.

DIRECT BONDING TO ENAMEL/ DENTIN WITH OPTIBOND ALL-IN-ONE -- SELF-ETCH TECHNIQUE

1. Prepare cavity. Wash thoroughly with water spray and air dry (Do not desiccate).
Note: For optimal adhesion, always prep (bevel) all enamel and/or margins prior to application of OptiBond All-In-One adhesive.
2. Shake OptiBond All-In-One adhesive bottle briefly.

Note: Upon first use, shake OptiBond All-In-One adhesive bottle vigorously for 10 seconds.

- Bottle Delivery: Dispense 2-3 drops of OptiBond All-In-One adhesive into a clean well. Replace cap immediately after dispensing.
 - Unidose® Delivery (INTENDED FOR SINGLE PATIENT USE ONLY): Open the Unidose® device and insert the applicator brush into the container to saturate the tip.
3. Using the disposable applicator brush, apply a generous amount of OptiBond All-In-One adhesive to the enamel/dentin surface. Scrub the surface with a brushing motion for 20 seconds.
 4. Apply a second application of OptiBond All-In-One adhesive with a brushing motion for 20 seconds.
 5. Dry the adhesive with gentle air first and then medium air for at least 5 seconds using oil-free air.
 6. Light cure.*
 7. Apply Herculite Ultra according to below instructions for use.

OPTIBOND SOLO PLUS and OPTIBOND FL

Herculite Ultra is also compatible for use with Kerr's total-etch bonding agents

OptiBond Solo Plus and OptiBond FL.
Follow manufacturer's instructions for appropriate technique guide.

PLACEMENT OF HERCULITE ULTRA COMPOSITE

Herculite Ultra is available in a carefully selected range of VITA® Enamel and Dentin Shades for true esthetic match yet simplified procedure. For details on all available shades and their various uses, please refer to "Explanation of Herculite Ultra Shading System" below for more information.

1. Select the desired shade.
 - **Syringe Delivery:** After extruding material onto pad, back off syringe at least two full turns to relieve pressure on the material.
 - **Unidose® Delivery:** Insert tip and properly align within dispenser. Place Unidose® tip at the deepest portion of the preparation.

Increments should be placed no more than 2mm at a time.

2. After placing each increment, stroke the composite to ensure marginal adaptation.
3. Light Cure each increment.*
When multiple surfaces are available, cure each surface for this recommended time.

EXPLANATION OF HERCULITE ULTRA SHADING SYSTEMS

Herculite Ultra is available in a variety of VITA® Enamel and Dentin Shades. Due to the extensive process involved in matching VITA® shades, many composite manufacturers choose to include their own shade guide with approximations of the actual VITA® shade. Since Kerr composites exhibit the ideal match to VITA®, simply use the VITA® shade guide in your office for selection.

(Note: VITA® is a registered trademark of VITA® Zahnfabrik Company, Germany)

For the complete list of available shades, go online to Kerrdental.com/composites

Enamel Shades may be used universally as a single body shade for most restorations. Herculite Ultra has the ability to transmit color from the tooth itself (chameleon effect) to match most tooth shades.

Dentin Shades may be used to match the opacity of dentin in deep restorations (Class I, II, IV).

XL Enamel Shade may be used to match bleached as well as deciduous teeth; it is lighter than the lightest VITA® Shade (B1).

Light Incisal Shade offers the dentist a high level of translucency for building incisal edges, or building translucent anatomy. It is an excellent overlay material on a variety of composite restorations.

A combination of XL Enamel and Light Incisal Shade offers true match to cosmetically whitened teeth and even veneers.

Unidose® Composite Gun Sterilization Guidelines - Steam Autoclave: 30 Minutes @ 270F.

* **Recommended Cure Times:** The Demi, 5 seconds, L.E.Demetron II, 5 seconds; L.E.Demetron I, 10 seconds; or Optilux 501 in Boost mode, 10 seconds, Ramp Mode, 20 seconds, or Regular Mode, 20 seconds. For all other lights, see manufacturer's recommendation.

CAUTION: OptiBond All-In-One adhesive is a corrosive liquid. Avoid contact with skin, eyes, and soft tissue. In case of contact with skin or eyes, flush immediately with water. Get medical attention for eyes. Do not take internally.

CAUTION: OptiBond All-In-One adhesive contains acetone and is flammable. Keep away from sources of ignition.

CAUTION: Uncured methacrylate resin may cause contact dermatitis and damage the pulp. Avoid contact with skin, eyes and soft tissue. Wash thoroughly with water after contact.

LIMITED WARRANTY

LIMITATION OF KERR'S LIABILITY

Kerr's technical advice, whether verbal or in writing, is designed to assist dentists in using Kerr's product.

The dentist assumes all risk and liability for damages arising out of the improper use of Kerr's product.

In the event of a defect in material or workmanship, Kerr's liability is limited, at Kerr's option, to replacement of the defective product or part thereof, or reimbursement of the actual cost of the defective product. In order to take advantage of this limited warranty, the defective product must be returned to Kerr. In no event shall Kerr be liable for any indirect, incidental, or consequential damages.

EXCEPT AS EXPRESSLY PROVIDED ABOVE, THERE ARE NO WARRANTIES, BY KERR, EXPRESS OR IMPLIED, INCLUDING WARRANTIES WITH RESPECT TO DESCRIPTION, QUALITY, OR FITNESS FOR A PARTICULAR PURPOSE.

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Kerr Corporation
Kerr Australia Pty. Ltd.
Sybron Dental Specialties Japan, Inc.