PULPDENT®

Etch-Royale™


Dental Etching Gel

• 37% H₃PO₄

 4 x 1.2 mL Etch-Royale

Rx only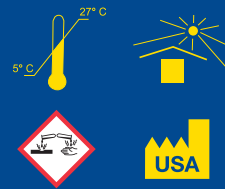

 PULPDENT Corporation • 80 Oakland Street • Watertown, MA 02472 • U.S.A.
www.pulpdent.com • pulpdent@pulpdent.com

PULPDENT®
Etch-Royale™
Dental Etching Gel

PULPDENT®

Etch Royale™ 37% PHOSPHORIC ACID

Dental Etching Gel • For Acid Etching Dentin, Enamel and Glass Ionomer Cement

Etch Royale is a thixotropic phosphoric acid etch gel which stays in place and will not run onto exposed tissue. Syringe dispensing through pre-bent small gauge applicator tips offers precise placement of gel even in relatively inaccessible areas. Etch Royale washes off easily with water.

DIRECTIONS FOR USE

1. Prepare tooth and cleanse the enamel surface to be etched with a slurry of pumice or non-oil containing prophylactic paste. Rinse and dry thoroughly.
2. Securely attach 25-gauge luer lock applicator tip to syringe. **Start flow on pad; not in patient's mouth. Do not push on plunger with thumb.** For safe, controlled dispensing, rest plunger in palm of hand and pull back on syringe barrel with thumb, index and middle fingers. Do not force plunger down into barrel.
3. Recommended etching time for enamel is 15-20 seconds and for dentin is 15 seconds. To the extent possible, apply Etch Royale first to enamel and then to dentin. Leave on dentin for 15 seconds. Cap syringe immediately after use.
4. Rinse off Etch Royale with copious amounts of water. Do not disturb this surface.
5. Leave surface wet, or dry surface with clean, uncontaminated air, depending on the instructions for use for the bonding system you are using.
6. Re-cap syringe immediately after use.

Note: Etched enamel should appear chalky white. If not, reapply etch gel to enamel for 20 seconds and rinse with copious amounts of water.

WARNING:

If Etch Royale contacts skin or soft tissue, rinse off immediately with copious amounts of water.

Note: Apply disposable barrier sleeves/wraps over multiple-use dental dispensers before use with each patient. For additional information, refer to: <http://www.fda.gov/MedicalDevices/ProductsandMedicalProcedures/DentalProducts/ucm404472.htm>

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