

Get what's next.

NexTemp!



NexTemp® Temporary Resin Cement

Get Firm Retention / Easy Removal

4X greater retention than most zinc oxide cements

Get Eugenol-Free

Non irritating to soft tissue / will not inhibit permanent cementation

Get Fluoride Release

Get Potassium Nitrate - Known to reduce sensitivity and act as a sedative to the pulp

Get Chlorhexidine - A proven anti-bacterial agent

Get Resin-Matrix Formulation

Good marginal seal with no wash-out

Two-Stage Curing

In approximately 2 minutes, initial gel set stabilizes temporary, allowing easy removal of excess. Total oral set time in 4-5 minutes.

Automix Syringe

Saves application time; guarantees consistent mix

Neutral Shade

Blends esthetically with temporary restoration / No show-through

EASY TO USE

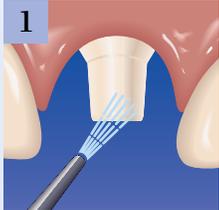
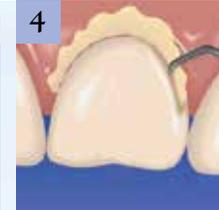
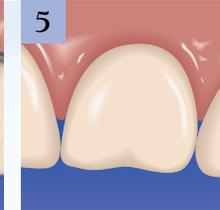
Less cement; more retention **Two-stage curing; easy clean up** **Firm retention***

* Total oral set time in 4-5 minutes

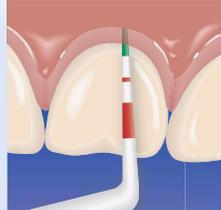


NexTemp® Temporary Cement Technique

PREMIER® NEXTEMP® TEMPORARY CEMENT IS EASY TO USE

- 1**  Dry and isolate the prepared tooth.
- 2**  Bleed the syringe. Mount a new automix dispensing tip and bleed the syringe again.
- 3**  Dispense a **small bead of cement around the inside margin.**
- 4**  Seat the temporary and hold in place. In approximately 2 minutes, excess cement reaches a distinct gel phase and can be removed safely.
- 5**  Final set occurs between 4-5 minutes after seating of temporary restoration.*

PERIOWISE®



Use a PerioWise® Periodontal Probe for post-treatment periodontal evaluation.

Standard - Item #3001471

Contains: One 5ml Automix Syringe, 10 Mix Tips

Bulk - Item #3001470

Contains: Four 5ml Automix Syringes

Syringe Tips - Item #3001451

Contains: 25 Automix Syringe Tips

* Final set time is determined intraorally. Ambient set times vary and are not representative of clinical conditions.

Flashing Crown Remover



Flashing Crown Remover • 1003576

As recommended by Dr. Donald Flashing



Flashing Crown Remover

Ideal for removing provisionally cemented crowns without damaging margins and is less traumatic to the patient.



Premier® Dental Products Company • 1710 Romano Drive, Plymouth Meeting, PA 19462, USA
 Phone 888-670-6100 • 610-239-6053 • Fax 610-239-6171 • www.premusa.com • E-mail: dentalinfo@premusa.com